Cochlear™

Wireless TV Streamer

User Manual

Manufacturer:

Cochlear Bone Anchored Solutions AB Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden
Tel: +46 31 792 44 00 Fax: +46 31 792 46 95

Regional offices:

Cochlear Ltd (ABN 96 002 618 073) 1 University Avenue, Macquarie University, NSW 2109, Australia Tel: +61 2 9428 6555 Fax: +61 2 9428 6352
Cochlear Americas 13059 E Peakview Avenue, Centennial, CO 80111, USA
Tel: +1 303 790 9010 Fax: +1 303 792 9025
Cochlear AG EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland
Tel: +41 61 205 0404 Fax: +41 61 205 0405

www.cochlear.com

94469

ACE, Advance Off-Stylet, AOS, AutoNRT, Autosensitivity, Beam, Clinicnet, Cochlear, Codacs, Contour, Contour Advance, Custom Sound, ESPrit, Freedom, Hear now. And always, Hybrid, inHear, Invisible Hearing, MP3000, myCochlear, NRT, Nucleus, Nucleus in Chinese characters, Off-Stylet, SmartSound, Softip, Sprint, the elliptical logo and Whisper are either trademarks or registered trademarks of Cochlear Limited. Ardium, Baha, Baha Divino, Baha Intenso, Baha PureSound, Baha SoftWear, DermaLock, Vistafix and WindShield are either trademarks or registered trademarks of Cochlear Bone Anchored Solutions AB. Dolby and the Dolby-D symbol are trademarks of Dolby Lab. © Cochlear Bone Anchored Solutions AB 2015. All rights reserved. FEB15.




613888-01

Hear now. And always


Cochlear®

Welcome

Congratulations on your purchase of a Cochlear™ Wireless TV Streamer.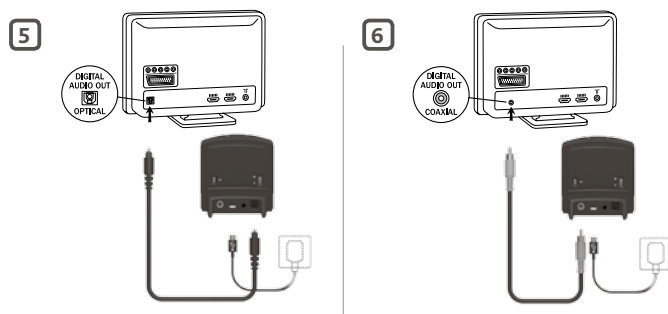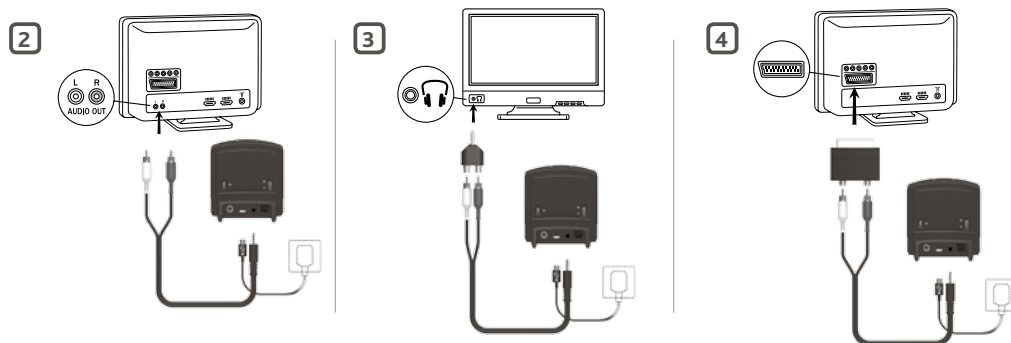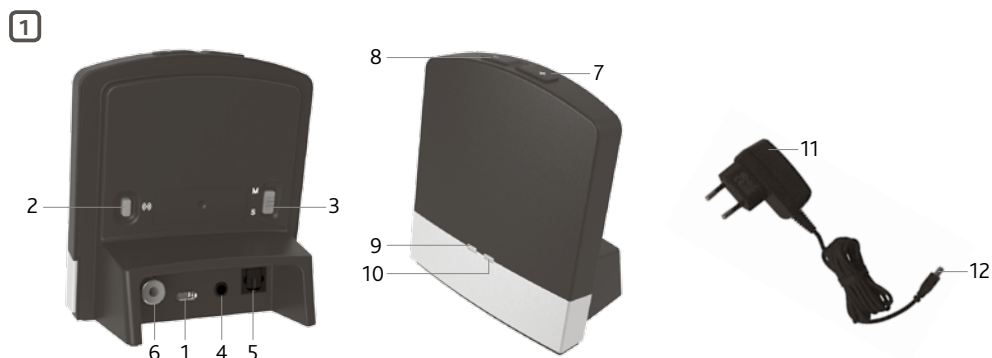

Cochlear Baha Remote Control**

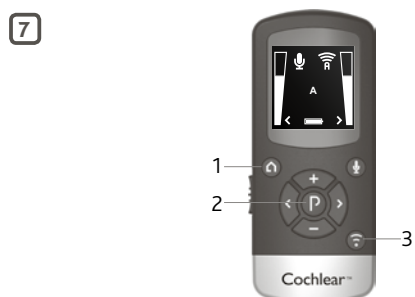Key to device *See figure 1*

1. Micro-USB power socket
2. Pairing button
3. Mono/stereo switch
4. Analogue mini-jack audio input
5. Optical (Toslink) audio input
6. Coaxial audio input
7. Increase volume
8. Decrease volume
9. Power light (green)
10. Activity light (yellow)
11. Wall plug
12. Micro-USB plug

Introduction

The Cochlear Wireless TV Streamer will provide you with high quality streamed audio from your TV, stereo, personal computer or similar device directly to your sound processors.

Please read this manual carefully to fully benefit from the Cochlear Wireless TV Streamer. Ask your hearing care professional if you have any questions. For further information you can also visit the www.cochlear.com web site.

For your safety, carefully read the chapter "Important Information".

Warranty

The warranty does not cover defects or damage arising from, associated with, or related to the use of this product with any non-Cochlear processing unit and/or any non-Cochlear implant. See "Cochlear Global Limited Warranty card" for more details.

Key to symbols

The following symbols will be used throughout this document.

	"Consult instructions for use" or "Operating instructions"
	"Caution" or "Caution, consult accompanying documents"
	CE-mark
	Serial Number
	Refer to instructions/booklet
	ACMA (Australian Communications and Media Authority) compliant
	Please ask your hearing care professional concerning disposal of this device.
	Wireless enabled

Getting Started

Installing the TV Streamer

Depending on your TV, stereo, computer or other audio setup, the installation of the TV Streamer may be done in several ways, which are described in the following section. The following steps will guide you through the pairing and connecting process. Start by connecting cables between the TV Streamer and the audio source and finish by pairing the TV Streamer to your sound processors.

Note: The TV Streamer supports both analogue and digital audio input. Please read the following sections for information about supported audio formats and how to connect the TV Streamer to audio sources.

Connecting the TV Streamer to an analogue audio source (e.g. TV or stereo):

See figure 2

1. Connect the power supply to the wall socket and insert the cable into the micro-USB socket on the TV Streamer.
2. Insert the male stereo mini-jack on the audio cable into the female stereo mini-jack input on the TV Streamer [no. 4 in figure 1].
3. Plug the other end audio cord into the "AUDIO OUT" terminals on the audio source. The "AUDIO OUT" is usually marked "L – R" for left and right, and can usually be found on the back of the audio source.

Note: With this installation, most audio sources will send sound to both sound processors and additional speakers connected to the audio source simultaneously. On some audio sources you can adjust the balance between the AUDIO OUT and the speakers. Consult your audio source manual for instructions.

Alternative analogue audio cable installations:

If it is not possible to connect the TV Streamer to the AUDIO OUTPUT of the audio source using the red and white plugs, the following installation alternatives can be used:

Mini-jack adaptor:

See figure 3

1. Connect the power supply to the wall socket and insert the cable into the micro-USB socket on the TV Streamer.
2. Insert the male stereo mini-jack on the audio cable into the female stereo mini-jack input on the TV Streamer [no. 4 in figure 1].
3. Connect the stereo mini-jack adaptor to the red and white plugs at the other end of the audio cable.
4. Plug the stereo mini-jack adaptor into the audio source's "HEADPHONE" output terminal or alternatively into your computer's audio terminal.

Note: The HEADPHONE output socket is usually located on the side or front of the audio source. When using the HEADPHONE socket, usually only the person wearing the sound processors will hear the sound and audio source speakers are muted. If muted speakers are not desired, please refer to the first installation, or the one in the following chapter. Your audio source may support speakers and HEADPHONE being on simultaneously; check your audio source manual for instructions.

Scart adapter:

See figure **4**

1. Connect the power supply to the wall socket and insert the cable into the micro-USB socket on the TV Streamer.
2. Insert the male stereo mini-jack on the audio cable into the female stereo mini-jack input on the TV Streamer [no. 4 in figure 1].
3. Connect the scart adapter to the red and white plugs at the other end of the audio cable.
4. Plug the scart adapter into the audio source's scart terminal. **Note: The scart output terminal is usually located on the back or side of the audio source.**

Please consult your audio source's manual if additional scart output configuration is needed.

Connecting the TV Streamer to a digital audio source (e.g. TV or stereo)

Note: The TV Streamer supports both analogue and digital audio input. When using digital audio input the following formats are supported:

Stereo PCM

- Sample rate: Up to 192 kHz
- Sample resolution: Up to 32 bit

Dolby Digital

- Channels: Max 5.1
- Bitrate: 32 kbs to 640 kbs
- Sample rate: 32, 44.1 and 48 kHz

Note: Dolby Digital Plus (requiring an HDMI interface), Dolby TrueHD, Dolby Pro Logic, Dolby Digital EX, Dolby Digital Surround EX, and AAC (Advanced Audio Coding) are not supported.

Using the optical (Toslink) audio input

See figure **5**

1. Connect the power supply to the wall socket and insert the cable into the micro-USB socket on the TV Streamer.
2. Insert one of the plugs on the optical audio cable into the optical audio input socket [no. 5 in figure 1] on the TV Streamer.

Note: Most optical cables are delivered with a small protective cap to protect the tip of the cable. Please remove this protective cap before inserting the cable.

3. Plug the other end of the optical audio cable into the "Digital Audio Out/ Optical" on your audio source.

Using the coaxial audio input

See figure **6**

1. Connect the power supply to the wall socket and insert the cable into the micro-USB socket on the TV Streamer.
2. Connect one of the plugs on the coaxial audio cable to the coaxial plug [no. 6 in figure 1] on the TV Streamer.
3. Plug the other end of the coaxial audio cable into the "Digital Audio Out/ Coaxial" on your audio source.

Pairing with your sound processors

You can pair up to three Cochlear streaming devices with your Cochlear sound processor.

Select a separate channel for each device paired with the sound processor.

Pairing the TV Streamer in channel 1

1. Make sure that the TV Streamer power supply is plugged into a wall socket and that the green light indicator on the front of the TV Streamer [no. 9 in

figure 1] is ON. Your sound processors must be OFF.

2. Press the pairing button on the TV Streamer [no. 2 in figure 1] once. The yellow indicator light on the front of the TV Streamer [no. 10 in figure 1] will now flash once every 2 seconds to indicate that the TV Streamer is ready to be paired in channel 1. Your TV Streamer will remain in pairing mode for 20 seconds.

3. While pairing mode is active, turn on your sound processor.

Successful pairing will be indicated by either an audible melody played in the sound processor, or by a flashing light on the sound processor (depending on the type of sound processor).

Pairing the TV Streamer in channel 2

1. Press the pairing button twice. The yellow indicator light on the front of the TV Streamer will now flash twice every 2 seconds to indicate that it is ready to be paired in channel 2.

2. While pairing mode is active, turn on your sound processor.

Pairing the TV Streamer in channel 3

1. Press the pairing button three times. The yellow indicator light on the front of the TV Streamer will now flash three times every 2 seconds to indicate that it is ready to be paired in channel 3.

2. While pairing mode is active, turn on your sound processor.

Pairing flashing pattern				
Pairing button pushes	LED blinking pattern			Channel
1	①	①	①	1
2	① ②	① ②	① ②	2
3	① ② ③	① ② ③	① ② ③	3

Streaming

Start streaming

Turn on the TV, stereo, computer or other audio device connected to the TV Streamer. Make sure your sound processors are turned on as well. Streaming may be started in one of two ways:

Option A

1. Press and hold the wireless streaming button on your sound processor for about three seconds.
2. An audible melody or a flashing light (depending on the type of sound processor) will be followed by streamed audio via the TV Streamer.

Option B

If you have a Cochlear Baha Remote Control** (optional), simply press the streaming button [no. 3 in figure 7] on this Baha Remote Control**. For more information, refer to the Baha Remote Control** manual.

Start streaming on a 2nd or 3rd TV Streamer unit See figure **7**

If your sound processor is paired with more than one streaming device, you can start streaming in two ways.

Option A

Press and hold the wireless streaming button on your sound processor for about three seconds a 2nd or 3rd time to access the 2nd or 3rd Cochlear streaming device.

Option B

Press the streaming button on a Baha Remote Control** a 2nd or 3rd time to access the 2nd or 3rd Cochlear streaming device.

The Cochlear streaming device(s) are indicated by the letters A, B or C, and the streaming device in use is shown on the Baha Remote Control** display.

Stop streaming

- Briefly press the streaming button on your sound processor. This will cancel streaming and return to the last program used.
- To stop streaming with the Baha Remote Control** press the "P" button, located in the center of the key pad.
- If you are using the telephone with Cochlear Wireless Phone Clip, streaming from the TV Streamer will be temporarily disrupted. When you have ended your phone call, streaming will resume.

Note: If you move out of your TV Streamer range (>7m) but return within the range before 5 minutes have passed, your sound processor will automatically reconnect and continue streaming.

Controls

Volume control

The TV Streamer is designed to provide a comfortable listening level from the moment it is activated. However, for some devices or listening environments, it may be desirable to adjust the listening volume and doing so can be accomplished in a variety of ways. Keep in mind that volume adjustments on the TV Streamer apply to the streamed signal only, not to the volume of normal environmental sounds. Use the "+" and "-" keys [no. 7 and 8 in figure 1] to adjust volume to a comfortable level. Make sure that your sound processors are comfortably adjusted before you change the settings with the volume control. In most cases this adjustment should only be made once since the TV Streamer will store the most recently set volume level. Use the Baha Remote Control 2** (optional) to adjust both the streaming volume level and the sound processor microphone volume level. Ask your hearing care professional for more information about this accessory. The volume can, in some cases, be further adjusted at the signal source, for example by turning the TV volume down. This will adjust the volume of the streamed sound transmitted wirelessly to the sound processors. Adjusting the TV volume will affect the listening volume in the room as well.

Adjusting sound processor and streaming volume is only possible with the Cochlear Baha Remote Control** and Cochlear Baha Sound Processor.

Mono/stereo control

The two-position switch [no. 3 in figure 1] may be used to set Stereo (two channels) or Mono (one channel) audio streaming. For binaural fittings, the switch should preferably be set to the Stereo position. For monaural fitting it should be set to the Mono position.

Adjustable delay mode

In rare cases there may be a misalignment between the audio streamed to the sound processors and the sound coming from the TV speakers (echo) or even a misalignment between the streamed audio and the TV picture (lip-sync). This may be evident in more complex audio setups where the TV Streamer is not connected directly to the TV (e.g. through a set-top box or other auxiliary unit). To overcome such misalignments the streaming delay from the TV Streamer can be adjusted as follows:

1. Make sure that the TV Streamer is turned on and that your sound processors are receiving the streamed audio.
2. Press and hold the pairing button [no. 2 in figure 1] for 3 seconds. The TV Streamer will then enter the adjustable delay mode where the green indicator light on the front of the TV Streamer will flash once every 2 seconds. The yellow indicator light will be on whenever an audio signal is present.
3. Use the "+" and "-" key [no. 7 and 8 in figure 1] on the top of the TV Streamer to increase or decrease the delay of the streamed signal in 10-ms steps. The streamed signal can be delayed in the range of 0-250 ms.

4. When the optimal alignment between the audio streamed to the sound processors and the sound coming from the TV speakers or alignment between the streamed audio and the TV picture has been reached, press and hold the pairing button for 3 seconds. Now the TV Streamer will resume normal operation, indicated by the green indicator light being constantly on.

The offset set in adjustable delay mode will be stored in the TV Streamer. If you do not exit the adjustable delay mode by pressing and holding the pairing button for 3 seconds, the TV Streamer will automatically resume normal operation after 30 minutes. To reset the delay to 0 ms (factory setting), press and hold the pairing button while simultaneously powering on the TV Streamer.

Important Information

Maintenance and Care

- Clean the TV Streamer using a damp cloth. Never use household cleaning products (washing powder, soap, etc.) or alcohol to clean the device.
- When the TV Streamer is not being used for extended periods, unplug the power supply from the wall socket.

General warnings

- Keep this device out of reach of children.
- The TV Streamer uses low-powered digitally coded transmissions in order to communicate with other wireless devices. Although unlikely, nearby electronic devices may be affected. In such cases, move the TV Streamer away from the affected electronic device.
- When using the TV Streamer and it is affected by electromagnetic interference, move away from the source of the interference.

Power supply warning

- Use the TV Streamer only with the power supply unit that was supplied with the product.
- The use of other power suppliers may damage or destroy the product and could be dangerous. Use of other power suppliers may invalidate any approval or void the warranty.
- When you disconnect the power cord or any accessory, grasp and pull the plug, not the cord.
- Never use a power supply that is damaged.
- Do not attempt to disassemble the power supply because it may expose you to a dangerous electric shock.
- Do not use the power supply outdoors or in damp areas.

General precautions

- Your sound processors and this device were given unique communication codes during fitting. This ensures that the device will not affect sound processors worn by others.
- High-powered electronic equipment, larger electronic installations and metallic structures may impair and significantly reduce the operating range.
- Use only original Cochlear accessories.
- Do not alter or modify this device.
- Opening the TV Streamer might damage it. If problems occur that cannot be resolved, consult your hearing care professional.
- The TV Streamer may only be repaired by an authorized service centre.
- Do not use the TV Streamer in areas where RF transmission is prohibited, e.g., in planes, etc.
- Only connect the TV Streamer to connections for which it is explicitly intended.

Technical specifications

Operating range	Up to 7 meters
Power supply	External wall adapter
Wireless	2,4 GHz
Operating temperature	0 to 55 °C
Storage temperature	-20 to 60 °C

Temperature test, transport and storage information

The product is subject to various tests in temperature and damp heating cycling between -25 °C and +70 °C according to internal and industry standards.

Wireless Accessory type designations for models included in this User Manual are:

FCC ID QZ3SAS-3, IC 8039C-SAS3, IC model: SAS-3

Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.
- Changes or modifications can void the user's authority to operate the equipment.

List of countries:

The products are in compliance with the following regulatory requirements:

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**** Note:** The Baha Remote Control is optional and works only with Baha Sound Processors.

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ID FCC QZ3SAS-3, IC 8039C-SAS3, modelo IC: SAS-3
-  SAS-3: MSIP-CRI-1CK-SAS-3