

The Baha 5 Power Sound Processor uses one Zinc-Air battery of size 675 (PR44). Refurbished sound processors are delivered without batteries.

The Baha 5 SuperPower Sound Processor uses a standard or compact rechargeable Lithium-Ion battery. A choice of two standard or compact rechargeable batteries, pre-selected at the time of order, are provided with each Baha 5 SuperPower Sound Processor.

Information about control platforms

There are two optional control platforms that can help control your Baha sound processor; the Baha Remote Control 2 and a compatible Android™ or iOS device with the Baha Smart App, or the Baha 5 Smart App. For compatibility information, visit www.cochlear.com/compatibility.

WARNING – Medical evaluation for people younger than 18:



P2058673

The use of a hearing aid in people younger than 18 years old without a medical evaluation may worsen impairment or disability. A prospective hearing aid user who is younger than 18 should have a recent medical evaluation from a doctor, preferably an ear-nose-throat doctor (an ENT). Prior to purchase, a doctor should determine that the person is a candidate for the use of a hearing aid.

Battery information inside

P2058673 D2058674-V2

Open for more information



WARNING: When to see a doctor

If you have any of the problems listed below, please see a doctor, preferably an ear-nose-throat doctor (an ENT).

- Your ear has a birth defect or an unusual shape. Your ear was injured or deformed in an accident.
- You saw blood, pus, or fluid coming out of your ear in the past 6 months.
- Your ear feels painful or uncomfortable
- You have a lot of ear wax, or you think something could be in your ear

- You get really dizzy or have a feeling of spinning or swaying (called vertigo)
- Your hearing changed suddenly in the past 6 months
- Your hearing changes: it gets worse then gets better again
- You have worse hearing in one ear
- You hear ringing or buzzing in only one ear

Note: Ask about trial-rental or purchase-option programs.

If you're unsure about your ability to get used to using a hearing aid, you should ask about a trial-rental or

purchase-option program. Many hearing instrument specialists offer programs that allow you to wear a hearing aid for a short time, at a nominal fee, before you decide to buy the hearing aid.

Cochlear™ Baha® sound processor battery information

The Cochlear™ Baha® 5 and Baha 6 Max sound processors use one Zinc-Air battery of size 312 (PR41).

One 6 piece package of size 312 Hearing Aid batteries is provided with each new Baha 6 Max Sound Processor.