Manufacturer:
Cochlear Bone Anchored Solutions AB Konstruktionsvägen 14, SE - 435 33 Mölnlycke, Sweden Tel: +46 31 792 44 00 Fax: +46 31 792 46 95

Regional offices:
Cochlear Ltd (ABN 96 002 618 073) 1 University Avenue, Macquarie University NSW 2109, Australia Tel: +61 2 9428 6555 Fax: +61 2 9428 6352
Cochlear Americas 13059 E Peakview Avenue, Centennial, CO 80111, USA Tel: +1 303 790 9010 Fax: +1 303 792 9025
Cochlear AG EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland Tel: +41 61 205 0404 Fax: +41 61 205 0405

Local offices:
Cochlear Deutschland GmbH & Co. KG Karl-Wiechert-Allee 76A, D-30625 Hannover, Germany Tel: +49 511 542 770 Fax: +49 511 542 7770
Cochlear Europe Ltd 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey KT15 2HJ, United Kingdom Tel: +44 1932 87 1500 Fax: +44 1932 87 1526
Nihon Cochlear Co Ltd Ochanomizu-Motomachi Bldg, 2-3-7 Hongo, Bunkyo-Ku, Tokyo 113-0033, Japan Tel: +81 3 3817 0241 Fax: +81 3 3817 0245
Cochlear (HK) Limited Unit 1810, Hopewell Centre, 183 Queens Road East, Wan Chai, Hong Kong SAR Tel: +852 2530 5773 Fax: +852 2530 5183
Cochlear Medical Device (Beijing) Co LtdUnit 2208-2212, Tower B, No.9 Germdale Building, No.91 Jianguo Road, Chaoyang District, Beijing, PR China 100022
P.R. CHINA Tel: +86 10 5909 7800 Fax: (+86 10) 5909 7900
Cochlear Limited (Singapore Branch) 6 Sin Ming Road, #01-16 Sin Ming Plaza Tower 2, Singapore 575585 Tel: +65 6553 3814 Fax: +65 6451 4105
Cochlear Korea Ltd 1st floor, Cheongwon building, 828-5, Yuksam dong, Kangnam gu, Seoul, Korea Tel: +82 2 533 4663 Fax: +82 2 533 8408
Cochlear Benelux NV Schaliënhoeverdreef 20 1, B - 2800 Mechelen, Belgium Tel: +32 1579 5511 Fax: +32 1579 5500
Cochlear Medical Device Company India (P) Ltd Platina Building, Ground Floor, Plot No. C-59, G-Block, BKC, Bandra (E), Mumbai – 400 0051, India
Tel: +91 22 6112 1111 Fax: +91 22 61121100
Cochlear Italia S.r.l. Via Larga n°33, 40138 Bologna, Italia Tel: +39 051 601 53 11 Fax: +39 051 39 20 62
Cochlear France S.A.S. Route de l'Orme aux Merisiers, Z.I. Les Algorithmes - Bât. Homère, 91190 Saint Aubin, France Tel: +33 811 111 993 Fax: +33 160 196 499
Cochlear Nordic AB Konstruktionsvägen 14, SE - 435 33 Mölnlycke, Sweden Tel: +46 31 335 14 61 Fax: +46 31 335 14 60
Cochlear Tıbbi Cihazlar ve Sağlık Hizmetleri Ltd. Şti. Cubuklu Mah. Bogazici Cad., Bogazici Plaza No: 6/1, Kavacik,TR - 34805 Beykoz-Istanbul, Turkey
Tel: +90 216 538 5900 Fax: +90 216 538 5919
Cochlear Canada Inc 2500-120 Adelaide Street West, Toronto, ON M5H 1T1 Canada Tel: +1 416 972 5082 Fax: +1 416 972 5083

www.cochlear.com


Baha, Baha Divino, Baha Intenso and Vistafix are registered trademarks of Cochlear Bone Anchored Solutions AB. Cochlear, Hear now. And always and the elliptical logo are either trademarks or registered trademarks of Cochlear Limited.
© Cochlear Bone Anchored Solutions AB 2013. All rights reserved. APR13. Printed in Sweden.

Cochlear™

Baha® Attract System

User manual part B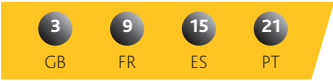

English Français Español Português

ZONE 8

Welcome

This guide includes tips and advice on how to best use and care for your Cochlear™ Baha® sound processor when using a magnet to connect it to your implant. By reading this manual and keeping it handy for future reference, you'll ensure that you get the most benefit out of your Baha sound processor.

NOTE: This manual is to be seen as a complement to the sound processor manual part A and does not replace it.

Introduction

This guide includes instructions on:

- Placing the SP magnet and sound processor
- Care, cleaning and maintenance
- Using accessories
- Troubleshooting

Note on figures: The figures included on the front and back covers correspond to sections of information in this manual which are common to all Baha sound processors. Please reference the appropriate figure when reading. The sound processor images shown are representative only and not to scale. The appearance of the sound processor included in your kit may vary.

Your kit includes the following items:

- Cochlear Baha sound processor
- Pack of batteries
- Safety line
- Share the Experience test rod
- Document pack (user manual, warranty card, registration card, MRI card)
- Abutment cleaning brush
(not to be used with magnetic retention system)
- Abutment cover
(not to be used with magnetic retention system)

The exact items depend on your specific type of processor.

Placing the SP magnet and the sound processor

See figure 

The SP magnet is a round disk that is attached to the sound processor to keep it in place over the implant under your skin and to transfer sound from the sound processor to the implant.

Use the tilt technique to safely and comfortably connect and disconnect your sound processor to the SP magnet, thereby reducing pressure on the snap connector and SP magnet.

To connect your sound processor:

- 1 Hold the SP magnet with one hand, or put it on a flat surface.
- 2 Hold the sound processor with the buttons positioned upwards, tilt it and gently snap it into place on the SP magnet.
- 3 Make sure that the marking on the SP magnet is facing upwards and place the SP magnet and sound processor over the implant behind your ear.

It may be easier to turn the processor on before placing it behind your ear.

To remove it from your head, grasp the SP magnet with the sound processor still attached.

- 4 To remove the sound processor from the SP magnet, place one finger under the sound processor and gently tilt until it releases.

Note:

- During the initial period of use it is recommended to gradually increase the time of use of the SP magnet to allow the skin to adapt to the pressure from the SP magnet. If any irritation is noted the SP magnet should be removed in order to let the skin rest for a while. If discomfort and/or skin irritation persist, contact your hearing care professional.
- Make sure that the marking on the SP magnet is always facing upwards to ensure good retention.
See figure  image 3.
- To avoid feedback (whistling), ensure that the sound processor is not touching any other items, such as glasses or a hat.
- It is recommended to wear a safety line attached to the sound processor to reduce the risk of loss or damage.
- For optimal retention, make sure that there is not too much hair underneath the SP magnet when placed over the implant.

SP magnet care

It is recommended to wear the SP magnet with the soft pad attached for maximum wearing comfort.

Keep your SP magnet clean by wiping it with an alcohol-free wipe. Do not use running water to clean your SP magnet.

Brush the snap coupling part clean using a soft brush.

Changing soft pads

You should change the SP Magnet soft pad regularly. You should replace the soft pad if you observe the following:

- An accumulation of dirt or moisture on the soft pad that cannot be wiped off
- The soft pad appears worn-out or damaged
- You experience a gradual deterioration in wearing comfort (if changing the soft pad does not help please contact your hearing care professional)

Contact your hearing care professional to obtain new soft pads.

See figure **2**

- 1 Remove the soft pad from the SP magnet, by lifting and gripping the unglued portion of the soft material, and then slowly release the soft pad from the SP magnet.
- Make sure that the adhesive remains are completely removed from the SP magnet.
- 2 Select a new soft pad and remove the cover sheet to expose the adhesive.
- 3 Attach the soft pad to the SP magnet. Make sure that the soft pad is centralised and covers the entire SP magnet surface.
- 4 Remove the remaining cover sheet from the soft pad before you start wearing the SP magnet.

Note:

- A small section on the edge of the soft pad is not covered by adhesive for easier grip and removal.

Supporting the processor

See figure **3**

Once you are familiar with your processor, you can change the program and the volume level while the sound processor is attached to the SP magnet placed on your head. Make sure you support the processor from the bottom with your thumb as you press the buttons with your index finger.

Sound processor care

Daily and weekly care is required to keep your sound processor functioning properly.

Long-term care

You will have regular check-ups as determined by your hearing care professional. These check-ups help ensure that the processor works and that your SP magnet fits properly.

Storing your processor

Store your sound processor in your storage box when it is not in use. The sound processor can be attached to the SP magnet and the safety line can be attached to the sound processor.

Cleaning the sound processor

See figure **4**

- 1 Clean the sound processor's housing with an alcohol-free wipe. Do not use water to clean your sound processor.
- 2 Brush the snap coupling clean using the soft brush. Avoid using excessive cleaning force on the snap coupling as this could damage the processor. Your hearing care professional can provide you with a special aftercare kit to keep your sound processor in good condition.

Connecting accessories via the Direct Audio Input (DAI)

See figure **5**

Match the accessory connector with the sound processor's DAI connector. While holding the sound processor gently at the corners, carefully push the accessory connector into the DAI port.

Dedicated DAI program and microphone mixing

An optional DAI program that is set by your hearing care professional allows you to listen exclusively to sounds through the DAI.

In all other programs, input from the DAI is combined with input from the microphone.

Using the telecoil

The telecoil unit improves sound when using some landline (fixed) and mobile telephones. It also gives you access to loop facilities in homes and public buildings. Attach the telecoil to the DAI port.

Using the Audio adapter

The Audio adapter allows direct input from computers, personal stereos, TVs, MP3 players and other external audio equipment. One end of the Audio adapter is connected to the sound processor at the DAI. The other end is connected to the audio device.

 **Caution: Connecting electrical equipment directly to the sound processor is extremely dangerous. Always use the Audio adapter when connecting an external audio source to your sound processor.**

Cochlear is not liable for any damage arising as a result of connecting unsuitable equipment or incorrect use.

Using an FM receiver

An FM receiver enables the use of FM systems that are commonly used in school classrooms.


Your hearing care professional can provide you with a list of available FM receivers. Attach your FM receiver to the DAI port. Keep extra batteries available if using an FM receiver as they have high energy consumption rates.

Warnings and precautions

 **Caution: If you are to undergo an MRI (Magnetic Resonance Imaging) procedure, refer to the MRI Reference Card included in the document pack.**

- If the SP magnet is too weak the sound processor may fall off. If it is too strong you may feel discomfort or experience soreness and irritation of the skin. If you are unsure about the strength of your SP magnet, contact your hearing care professional.
- If you experience persistent irritation or soreness of the skin, or if the SP magnet gives you any discomfort, remove it and contact your hearing care professional.
- If you wear hats, etc. be sure to avoid pressure on your SP magnet as this may cause irritation to the tissue under this magnet.
- Do not wear your sound processor and SP magnet while sleeping as this may cause irritation to the skin under the SP magnet.
- Keep the SP magnet in a safe place and keep it away from magnetic cards (e.g. credit cards, bus cards, etc.) as these may be damaged by the magnet.

- Remember that your implant contains a magnet, so be aware that it may attract other metallic objects that are held in close proximity (e.g. hairdresser's scissors, etc.).
- Do not wear any other magnets (e.g. fridge magnets) over your implant than SP magnets from Cochlear supplied to you by your hearing care professional.
- Do not expose the SP magnet to extreme temperatures. The SP magnet is designed to operate within the temperature ranges +5°C (+41°F) to +40°C (+104°F). The SP magnet should not be subject, at any time, to temperatures below -20°C (-4°F) or above +50°C (+122°F).
- This product is not suitable for use in flammable and/or explosive environments.

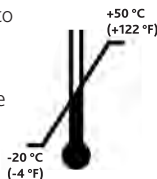- **If the SP magnet and soft material becomes soaked with water:** Put it in a container with drying capsules such as a Dri-aid kit or similar. Leave it to dry out overnight. Drying kits are available from most hearing care professionals.
- **For parents or caregivers:** regularly check to see if the user experiences persistent irritation or soreness of the skin, or if the SP magnet causes any discomfort. If so remove it and contact the hearing care professional.

 **Caution:** Batteries can be harmful if swallowed. Be sure to keep your batteries out of reach of small children, those with mental challenges, and household pets. In the event a battery is accidentally swallowed, seek immediate medical attention at the nearest emergency centre.

Troubleshooting

Battery function can be related to a number of problems, such as no sound, intermittent sound and crackling/buzzing. A new battery is often the solution.

If the possible solutions in this chart do not solve your problem, please see your hearing care professional for further assistance.

Problem	Cause	Possible Solution
No sound/weak sound	Weak or dead battery.	Replace battery.
	Volume too low.	Turn up volume.
	Magnet is too weak.	Contact your hearing care professional.
Feedback (whistling)	Hat, glasses or other object in contact with sound processor.	Relocate or remove the object.
	Battery door in wrong position.	Close battery door.
Sound is distorted or intermittent	Too high volume.	Decrease volume.
	Weak battery.	Replace battery.
Sound processor repeatedly falls off	Magnet is too weak.	Contact your hearing care professional.
Persistent skin irritation	Magnet is too strong.	Contact your hearing care professional.
Sound processor does not work	Sound processor not turned on.	Turn on sound processor.
	Battery door is not completely closed.	Gently close the battery door completely.
	Dead battery.	Replace battery.
	Battery inserted wrong way.	Make sure battery is inserted correctly.

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