

POCKET GUIDE

COCHLEAR™ TRUE WIRELESS™ MINI MICROPHONE 2 & 2+



Mini Microphone 2



Mini Microphone 2+



Cochlear®

Welcome to Cochlear™ True Wireless™ Technology. **No Strings Attached.**

Index

- 3. Overview
- 4. Getting Started
- 5. Product Diagram
- 6. Pairing
- 8. Using
- 11. Care & Maintenance

Visit us online for helpful tips,
how-to videos and more:

www.Cochlear.com/US/N6wireless
Recipients.MyCochlear.com


The Cochlear™ Wireless Mini Microphone 2 & 2+ are designed to allow you to hear better over a distance and in noisy listening situations, such as:

- In meetings
- When traveling by car, bus or train
- Dining at a restaurant
- Participating in a class at the gym

How to use:

Give the Mini Microphone to the person you want to listen to and ask them to position it close to their mouth by placing it vertically on their clothing using the clip on the back. Then activate streaming on your CP900 Series Sound Processor.

Place your Mini Microphone on a table to listen to multiple speakers in a group/conference setting (*2+ only*).

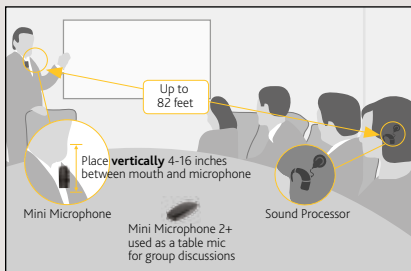

Getting Started

Charging the battery

Plug one end of the included micro USB charging cable into the Mini Microphone and the other into a computer, or into a power outlet via the supplied adapter.

When the LED indicator on top of the Mini Microphone turns green with a red blink every 2 seconds charging is complete.

Before you use your Mini Microphone you should allow it to charge for at least 3 hours, even if the indicator light shows a full charge. The battery cannot be overcharged.

Note: *On average, the Mini Microphone will operate for up to 10 hours between charges.*

Product Diagram



Pairing


Follow the four steps below to pair your Nucleus® 6 Sound Processor to the Cochlear Wireless Mini Microphone

1 Switch off your CP900 Series Sound Processor.

*Remove the Sound Processor from your ear.
Disconnect the battery.*

OR


Press and hold Kanzo button until the light is a steady orange.

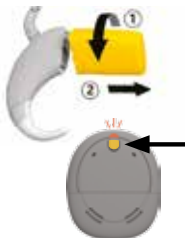

2 Press and hold the ON/OFF button on the Mini Microphone for 2 seconds until the LED turns green, then release.



- 3** Press the pairing button on the (A) Mini Microphone 2 or (B) Mini Microphone 2+ once using the tip of a pen or similar object.

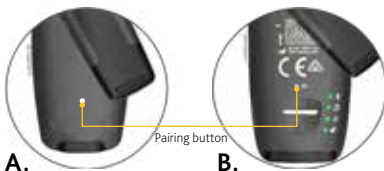

Note: The LED will flash yellow every 2 seconds and the Mini Microphone will now be in pairing mode for 20 seconds.

- 4** While pairing mode is active (20 seconds), attach the battery to the Nucleus 6 Sound Processor (CP910/920) or Press the button on the Kanso Sound Processor (CP950) to turn on.



A blue indicator light flashes to show that pairing has been successful.

Have more than one Wireless Accessory?
Consult the User Guide to pair up to three
devices to your sound processor(s).

Using

Now that the Cochlear Wireless Mini Microphone is paired you are ready to start streaming. There are several ways to do this.

Start streaming using your sound processor

- 1 Turn on your sound processor and Mini Microphone.**

Press and hold the upper button on your sound processor for 2 seconds, then release.*

The sound processor flashes a blue light to indicate audio is streaming.

**Not available if sound processor buttons are in advanced mode.*

OR

Press† and hold button for 2 seconds then release to stream audio.

†Press and release again if you need to cycle to the next audio source. This is necessary if your audiologist enabled the Telecoil as this is the first streaming audio source.



2 To stop streaming.

Tap the upper button.



Tap the button.



Start streaming using your CR210 Remote Control

1 Turn on your sound processor and Mini Microphone. Turn on your CR210 Remote Control.

Press and hold the Telecoil button for 2 seconds, then release.

Audio will start streaming through the Mini Microphone.



2 To stop streaming.

Tap the Telecoil button.



Start streaming using your CR230 Remote Assistant

Method 1: Turn on your sound processor and Mini Microphone. Turn on your CR230 Remote Assistant.

Press and hold the Telecoil button for 2 seconds, then release.

Audio will start streaming through the Mini Microphone.

To stop streaming, tap the Telecoil button.



Method 2: Turn on your sound processor and Mini Microphone. Turn on your CR230 Remote Assistant.

From the Home screen, press the right arrow button until you see the Stream screen.

Scroll down the list to the channel you want to use and press OK.



When your audio device is turned on, audio will start streaming through the Mini Microphone.

To stop streaming, select Off on the Stream selection screen.

Care and Maintenance

- Clean the Cochlear Mini Microphone using a damp cloth. Never use household cleaning products (*washing powder, soap, etc.*) or alcohol to clean the device.
- When the Cochlear Mini Microphone is not in use, turn it off and store it safely.
- Protect the Cochlear Mini Microphone from excessive moisture (*bathing, swimming*) and heat (*radiator, car dashboard*).
- Protect it from excessive shock and vibration.
- Protect the microphone and charger sockets from debris and dirt.

Visit us online for helpful tips,
how-to videos and more:

www.Cochlear.com/US/N6wireless
Recipients.MyCochlear.com

www.Cochlear.com/US

Cochlear Americas

13059 East Peakview Avenue
Centennial, CO 80111 USA
Telephone: 1 303 790 9010
Support: 1 800 483 3123

Cochlear Canada Inc.

2500-120 Adelaide Street West
Toronto, ON M5H 1T1 Canada
Telephone: 1 416 972 5082
Fax: 1 416 972 5083

©Cochlear Limited 2016. All rights reserved. Trademarks and
registered trademarks are the property of Cochlear Limited.



FUN2593 ISS2 OCT16