Manufacturer:
Cochlear Bone Anchored Solutions AB Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden Tel: +46 31 792 44 00 Fax: +46 31 792 46 95

Regional offices:
Cochlear Ltd (ABN 96 002 618 073) 1 University Avenue, Macquarie University, NSW 2109, Australia Tel: +61 2 9428 6555 Fax: +61 2 9428 6352
Cochlear Americas 13059 E Peakview Avenue, Centennial, CO 80111, USA Tel: +1 303 790 9010 Fax: +1 303 792 9025
Cochlear AG EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland Tel: +41 61 205 0404 Fax: +41 61 205 0405

Local offices:
Cochlear Deutschland GmbH & Co. KG Karl-Wiechert-Allee 76A, 30625 Hannover, Germany Tel: +49 511 542 770 Fax: +49 511 542 7770
Cochlear Europe Ltd 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey, KT15 2HJ, United Kingdom Tel: +44 1932 26 3400 Fax: +44 1932 26 3426
Nihon Cochlear Co Ltd Ochanomizu-Motomachi Bldg, 2-3-7 Hongo, Bunkyo-Ku, Tokyo 113-0033, Japan Tel: +81 3 3817 0241 Fax: +81 3 3817 0245
Cochlear (HK) Limited Unit 1204, 12/F, 297-307 Hennessy Road, Wan Chai, Hong Kong Tel: +852 2530 5773 Fax: +852 2530 5183
Cochlear Medical Device (Beijing) Co Ltd Unit 2208-2212, Germdale Tower B, 91 Jianguo Road, Chaoyang District, Beijing 100022, PR China
Tel: +86 10 5909 7800 Fax: +86 10 5909 7900
Cochlear Limited (Singapore Branch) 6 Sin Ming Road, #01-16 Sin Ming Plaza Tower 2, Singapore 575585 Tel: +65 6553 3814 Fax: +65 6451 4105
Cochlear Korea Ltd 1st floor, Cheongwon building, 828-5, Yuksam dong, Kangnam gu, Seoul, Korea Tel: +82 2 533 4663 Fax: +82 2 533 8408
Cochlear Benelux NV Schaliënhoevedreef 20 I, 2800 Mechelen, Belgium Tel: +32 1579 5511 Fax: +32 1579 5500
Cochlear Medical Device Company India PVT Ltd Platina Bldg, Ground Floor, Plot No. C 59, G Block, BKC, Bandra East, Mumbai 400051, India
Tel: +91 22 6112 1111 Fax: +91 22 61121100
Cochlear Italia SRL Via Larga 33, 40138 Bologna, Italy Tel: +39 051 601 53 11 Fax: +39 051 39 20 62
Cochlear France S.A.S. Route de l'Orme aux Merisiers, ZI les Algorithmes – Bât. Homère, 91190 Saint-Aubin, France Tel: +33 805 200 016 Fax: +33 160 196 499
Cochlear Nordic AB Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden Tel: +46 31 335 14 61 Fax: +46 31 335 14 60
Cochlear Tıbbi Cihazlar ve Sağlık Hizmetleri Ltd. Sti. Cubuklu Mah. Bogazici Cad. Bogazici Plaza No: 6/1 Kavacik, 34805 Beykoz-Istanbul, Turkey
Tel: +90 216 538 5900 Fax: +90 216 538 5919
Cochlear Latinoamerica, S. A. International Business Park Building 3835, Office 103, Panama Pacifico, Panama Tel: +507 830 6220 Fax: +507 830 6218
Cochlear Canada Inc 2500-120 Adelaide Street West, Toronto, ON M5H 1T1, Canada Tel: +1 416 972 5082 Fax: +1 416 972 5083

www.cochlear.com

Ardium, Baha, Baha Divino, Baha Intenso, Baha PureSound, Baha SoftWear, DermaLock, Vistafix and WindShield are either trademarks or registered trademarks of Cochlear Bone Anchored Solutions AB. Cochlear, Hear now. And always and the elliptical logo are either trademarks or registered trademarks of Cochlear Limited.
© Cochlear Bone Anchored Solutions AB 2015. All rights reserved. JAN15. Printed in Sweden.

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Hear now. And always

Welcome

Congratulations on getting your new Baha® sound processor. This manual provides tips and advice on how to best use and care for your Cochlear™ Baha® Attract System. It is a complement to the sound processor manual part A and does not replace it.

Introduction

This guide includes instructions on:

- attaching your sound processor to the SP magnet
- care, cleaning and maintenance
- using accessories
- troubleshooting

Note on figures: The figures included on the front and back covers correspond to sections of information in this manual which are common to all Baha sound processors. Please reference the appropriate figure when reading. The sound processor images shown are representative only and not to scale. The appearance of the sound processor included in your kit may vary.

Attaching your sound processor and SP magnet

See figure **1**

The SP magnet attaches to your sound processor to keep it in place over the implant magnet and to transfer sound from your sound processor to the implant.

Use the tilt technique (described below) to connect and disconnect your sound processor from the SP magnet. This will ensure it remains in good working order.

- 1 Place the SP magnet on a flat surface.
- 2 Gently tilt and snap the sound processor into the SP magnet coupling. Ensure you align the button(s) on the sound processor with the arrow mark on the SP magnet.

- 3 With the button(s) and arrow mark facing upwards, place the sound processor over the implant behind your ear. It may be easier to turn the sound processor on before you place it behind you ear.
- 4 Remove the sound processor by grasping the SP magnet with the sound processor still attached. To remove the sound processor from the SP magnet, place one finger under the sound processor and gently tilt until it releases.

Note:

- During the initial period of use it is recommended to gradually increase the time of use of the SP magnet to allow the skin to adapt to the pressure from the SP magnet. If any irritation is noted the SP magnet should be removed in order to let the skin rest for a while. If discomfort and/or skin irritation persist, contact your hearing care professional.
- To ensure good retention the arrow on the SP magnet should always face upwards and you should try to limit the amount of hair underneath the SP magnet.
See figure **1** image 3.
- To avoid feedback (whistling), ensure that the sound processor is not touching any other items, such as glasses or a hat.
- It is recommended to wear a safety line attached to the sound processor to reduce the risk of loss or damage.

Caring for your SP magnet

Keep your SP magnet clean by wiping it with an alcohol-free wipe. Do not use running water to clean your SP magnet.

Clean the SP magnet coupling using a soft brush.

It is recommended to wear the SP magnet with the soft pad attached for maximum wearing comfort.

Changing soft pads

You should change the SP magnet soft pad regularly. You should replace the soft pad if you observe the following:

- An accumulation of dirt or moisture on the soft pad that cannot be wiped off
- The soft pad appears worn-out or damaged
- You experience a gradual deterioration in wearing comfort (if changing the soft pad does not help please contact your hearing care professional)

Contact your hearing care professional to obtain new soft pads.

See figure **2**

- 1 Remove the soft pad from the SP magnet, by lifting and gripping the unglued portion of the soft material, and then slowly release the soft pad from the SP magnet.

Make sure that the adhesive remains are completely removed from the SP magnet.

- 2 Select a new soft pad and remove the cover sheet to expose the adhesive.
- 3 Attach the soft pad to the SP magnet. Make sure that the soft pad is centralised and covers the entire SP magnet surface.
- 4 Remove the remaining cover sheet from the soft pad before you start wearing the SP magnet.

Note: A small section on the edge of the soft pad is not covered by adhesive for easier grip and removal.

Handling your sound processor

See figure **3**

Once you are familiar with your processor, you can change the program and the volume level while the sound processor is attached to the SP magnet placed on your head. Make sure you support the processor from the bottom with your thumb as you press the buttons with your index finger.

Storing your processor

Store your sound processor in your storage box when it is not in use. The sound processor can be attached to the SP magnet and the safety line can be attached to the sound processor.

Wireless Accessories

See figure **4**

With Baha sound processors you can use the Cochlear Wireless Accessories to enhance your listening experience. Ask your hearing care professional to learn more about your options or visit www.cochlear.com. Please note the Cochlear Wireless Accessories are not compatible with all sound processor models.

Other Audio Accessories

Connecting accessories via the Direct Audio Input (DAI)

- 1 Match the accessory connector with the sound processor DAI connector.
- 2 Holding the sound processor gently at the corners, carefully push the accessory connector into the DAI port.

Note: Not all sound processors have a Direct Audio Input (DAI) access. Consult your user manual part A for more information.

Dedicated DAI program and microphone mixing

An optional DAI program that is set by your hearing care professional allows you to listen to sounds through the DAI.

Using the telecoil

The telecoil unit improves sound when using some landline (fixed) and mobile telephones. It also gives you access to loop facilities in homes and public buildings. Attach the telecoil to the DAI port.

Using the Audio adapter

The Audio adapter allows direct input from computers, personal stereos, TVs, MP3 players and other external audio equipment. One end of the Audio adapter is connected to the sound processor at the DAI. The other end is connected to the audio device.

 **Caution:** Connecting electrical equipment directly to the sound processor is extremely dangerous. Always use the Audio adapter when connecting an external audio source to your sound processor.

Cochlear is not liable for any damage arising as a result of connecting unsuitable equipment or incorrect use.

Using an FM receiver

An FM receiver enables the use of FM systems that are commonly used in school classrooms.

Your hearing care professional can provide you with a list of available FM receivers. Attach your FM receiver to the DAI port. Keep extra batteries available if using an FM receiver as they have high energy consumption rates.

Warnings and precautions

Caution: If you are to undergo an MRI (Magnetic Resonance Imaging) procedure, refer to the MRI Reference Card included in the document pack, or the information below.

MRI Safety Information

The sound processor and SP Magnet must be removed before entering a room where an MRI scanner is located.

Non-clinical testing has demonstrated that the BIM400 Implant Magnet, in combination with a BI300 Implant, is MR Conditional. It can be scanned safely under the following conditions. Scanning under other conditions may result in severe patient injury or device malfunction.

- Static magnetic field of 1.5 Tesla only
- Maximum spatial gradient field of 26600 Gauss/cm (266 T/m)
- Maximum switched gradient slew rate per axis of 200 mT/m/ms
- Maximum switched gradient amplitude per axis of 45 mT/m
- Maximum MR System reported whole body averaged specific absorption rate (SAR) of 2.0 W/kg (Normal Operating Mode)

Additional instructions essential to safe use in the MR environment:

- Under the scan conditions defined above, the BIM400 Implant Magnet is expected to produce a maximum temperature rise of 2.1 ° C after 15 minutes of continuous scanning.
- In non-clinical testing, the BIM400 Implant Magnet produced a temperature rise of less than 2.1 ° C (extrapolated) at a maximum whole body averaged specific absorption rate (SAR) of 2.0 W/kg (extrapolated) assessed by calorimetry for 15 minutes of MR scanning in a 1.5 Tesla Intera, Philips Medical Systems (Software: 12.6.1.3, 2010-12-02) MR Scanner.
- In non-clinical gradient-induced heating testing, the BIM400 Implant Magnet produced a temperature rise (extrapolated) of less than 4.5 ° C at a time rate of change of the theoretical maximum worst-case gradient magnetic field dB/dt (extrapolated) of 200 T/s during 30 min. of continuous exposure in a test laboratory system (Pulsed Magnetic Field Generator) equivalent with a gradient system of a 1.5 Tesla MR system.
- In non-clinical testing with the implant magnet in place, the image artefact caused by the device extends approximately 11.5 cm (4.5 in.) from the BIM400 Implant Magnet when imaged with a gradient echo pulse sequence and a 1.5 Tesla MRI system.

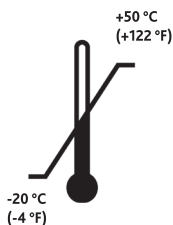
Other Safety Information

- Remove the sound processor and SP Magnet for any treatment involving ionizing radiation, such as for example X-ray, CT-scan and radiation therapy.
- The implanted magnet may affect diagnostic information/treatment effectiveness in the area around the magnet during a treatment with ionizing radiation (e.g. X-Ray, CTscan, radiation therapy). Consult your treating physician before going through any such treatment.
- The Baha Attract System contains small parts that may be hazardous if swallowed or may cause choking if ingested or inhaled.
- The Baha Attract System contains magnets that should be kept away from life supporting devices, e.g. cardiac pacemakers and ICDs (implantable cardioverter defibrillators) and magnetic ventricular shunts, as the magnets may affect the function of these devices. Contact the manufacturer of the specific device to find out more.
- The Baha Attract System (SP Magnet and implanted magnet) contains magnets that may interfere with or damage surrounding electronic devices, for example mobile phones.
- If the SP magnet is too weak the sound processor may fall off. If it is too strong you may feel discomfort or experience soreness and irritation of the skin. If you are unsure about the strength of your SP magnet, contact your hearing care professional.

- If you experience persistent irritation or soreness of the skin, or if the SP magnet gives you any discomfort, remove it and contact your hearing care professional.
- If you wear hats, etc. be sure to avoid pressure on your SP magnet as this may cause irritation to the tissue under this magnet.
- Do not wear your sound processor and SP magnet while sleeping as this may cause irritation to the skin under the SP magnet.
- The Baha Attract System (SP Magnet and implanted magnet) contains magnets that may damage magnetic cards (e.g. credit cards, bus cards, etc.) and data storage devices.
- Remember that your implant contains a magnet, so be aware that it may attract other metallic objects that are held in close proximity (e.g. hairdresser's scissors, etc.).
- Do not wear any other magnets (e.g. fridge magnets) over your implant than SP magnets from Cochlear supplied to you by your hearing care professional.
- Do not expose the SP magnet to extreme temperatures. The SP magnet is designed to operate within the temperature ranges $+5^{\circ}\text{C}$ ($+41^{\circ}\text{F}$) to $+40^{\circ}\text{C}$ ($+104^{\circ}\text{F}$). The SP magnet should not be subject, at any time, to temperatures below -20°C (-4°F) or above $+50^{\circ}\text{C}$ ($+122^{\circ}\text{F}$).
- This product is not suitable for use in flammable and/or explosive environments.
- **If the SP magnet and soft material becomes soaked with water:** Put it in a container with drying capsules such as a Dri-aid kit or similar. Leave it to dry out overnight. Drying kits are available from most hearing care professionals.
- **For parents or caregivers:** regularly check to see if the user experiences persistent irritation or soreness of the skin, or if the SP magnet causes any discomfort. If so remove it and contact the hearing care professional.

Caution: Batteries can be harmful if swallowed. Be sure to keep your batteries out of reach of small children and household pets. In the event a battery is accidentally swallowed, seek immediate medical attention at the nearest emergency centre.

Troubleshooting

Battery function can be related to a number of problems, such as no sound, intermittent sound and crackling/buzzing. A new battery is often the solution.

If the possible solutions in this chart do not solve your problem, please see your hearing care professional for further assistance.

Problem	Cause	Possible Solution
No sound/weak sound	Weak or dead battery.	Replace battery.
	Volume too low.	Turn up volume.
	Magnet is too weak.	Contact your hearing care professional.
Feedback (whistling)	Hat, glasses or other object in contact with sound processor.	Relocate or remove the object.
	Battery door in wrong position.	Close battery door.
Sound is distorted or intermittent	Too high volume.	Decrease volume.
	Weak battery.	Replace battery.
Sound processor repeatedly falls off	Magnet is too weak.	Contact your hearing care professional.
Persistent skin irritation	Magnet is too strong.	Contact your hearing care professional.
Sound processor does not work	Sound processor not turned on.	Turn on sound processor.
	Battery door is not completely closed.	Gently close the battery door completely.
	Dead battery.	Replace battery.
	Battery inserted wrong way.	Make sure battery is inserted correctly.
	Key lock is activated.	Deactivate key lock.

Additional Information

Instructions for use in the USA

Caution: Federal law (USA) restricts this device to sale or by the order of a medical practitioner.

Warning to hearing aid dispensers

A prospective Baha user must always have a medical evaluation by a licensed physician (preferably an ear specialist) and have an implant placed in the skull bone before a Baha sound processor can be dispensed. The Baha sound processor can be used with a softband which does not require an implant. However, a medical evaluation is still warranted.

Important notice for prospective users of a Baha sound processor

- A prospective Baha user must always have a thorough medical evaluation by a licensed physician (preferably a physician who specializes in diseases of the ear, otolaryngologist, otologist, or otorhinolaryngologist) and have an implant placed in the skull bone before a Baha sound processor is purchased.
- The purpose of the medical evaluation is to ensure that all medically treatable conditions that may affect hearing are identified and treated before the sound processor is purchased. Following the medical evaluation, the physician will give you a written statement that states that your hearing loss has been medically evaluated and that you may be considered a Baha candidate. The physician will refer you to an audiologist or a hearing aid dispenser, as appropriate, for an audiological evaluation.

- Federal law restricts the sale of Baha sound processors to those individuals who have obtained a medical evaluation from a licensed physician. Federal law permits a fully informed adult to sign a waiver statement declining the medical evaluation for religious or personal beliefs that preclude consultation with a physician. The exercise of such a waiver is not in your best health interest and its use is strongly discouraged.

A word of caution about batteries

Batteries can be harmful if swallowed. Be sure to always keep your batteries out of reach of small children, pets, or people of diminished capacity. In the event a battery is accidentally swallowed, seek prompt medical attention at the nearest emergency center, or call the National Button Battery Hotline collect at (202) 625 – 3333.

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