

Cochlear™

Quick Guide to the Nucleus® CR210 Remote Control

- ☒ Getting started
- ☒ Quick reference



Hear now. And always



About

Use the Cochlear™ Nucleus® CR210 Remote Control to control the commonly used functions of your Cochlear™ Nucleus® CP910 or CP920 Sound Processor.


NOTE: More information and warnings in the *CR210 Remote Control User Guide*.



CR210 Remote Control



ABOUT

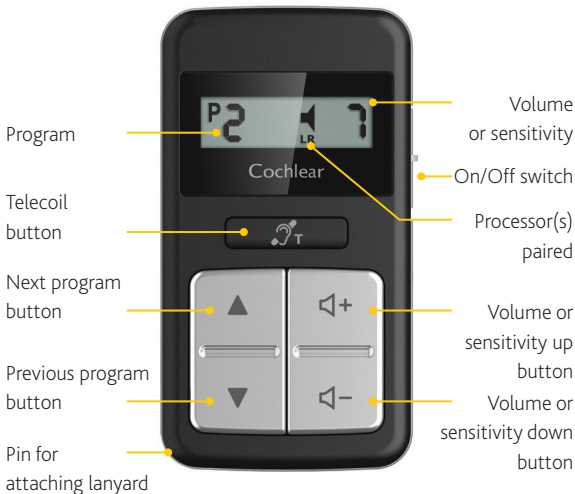

Change battery

Change the battery whenever you see the low battery indicator.



When display shows low battery in remote control:



Unscrew cover with a Phillips head (cross-head) screwdriver.



Remove cover and old battery.



CAUTION

Use ONLY disposable CR2032 or 5004LC 3V standard lithium coin cell batteries in your remote control.



Carefully insert
new battery,
plus side "+" up.



Replace
cover, bottom
edge first.



Screw cover
back on.



BATTERY

Turn on

Slide the switch
up to turn on.



Turn off

Slide the switch
down to turn off.



TURN ON

Pair

Pairing allows the remote control to communicate with the processor.



When display shows remote control is not paired:



Turn on both your processor and remote control.

Place coil on the back of your remote control.



Remote control shows pairing in progress.



Remote control shows it's paired.



If you have a second processor, repeat steps 2–4.



TURN ON

Change programs

Next program

Previous program



Adjust volume/sensitivity



Turn up either
volume or
sensitivity
(as set up by
your clinician)

Turn down
volume or
sensitivity



USE

Use telecoil

Listen to hearing loops and hear voices on the telephone with telecoil or auto telecoil. Auto telecoil detects the telecoil signal automatically.

Turn telecoil on:




Press once.

Turn telecoil
to auto:



Press twice.

Turn telecoil off:

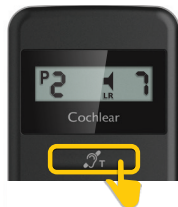

Press once.

Use audio accessories

When an audio accessory is connected to your processor, use your remote control to turn the accessory off and back on.



USE

Display shows
audio accessory:



Turn audio
accessory off:



Press once.

Turn audio
accessory on:



Press once.

Care

Clean your remote control with a soft dry cloth.
Keep your remote control dry.





QUICK GUIDE



Cochlear Ltd (ABN 96 002 618 073)

1 University Avenue, Macquarie University, NSW 2109, Australia

Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

Cochlear Ltd (ABN 96 002 618 073)

14 Mars Road, Lane Cove, NSW 2066, Australia

Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

Cochlear Americas

13059 E Peakview Avenue, Centennial, CO 80111, USA

Tel: +1 303 790 9010 Fax: +1 303 792 9025

ECREP Cochlear Deutschland GmbH & Co. KG

Karl-Wiechert-Allee 76A, 30625 Hannover, Germany

Tel: +49 511 542 770 Fax: +49 511 542 7770

www.cochlear.com

Cochlear, Hear now. And always, Nucleus, and the elliptical logo
are either trademarks or registered trademarks of Cochlear Limited.

© Cochlear Limited 2013

Hear now. And always

402610 ISS6 SEP13

Printed in Australia