Cochlear™

Baha® Cordelle II

User manual part A

- 3 GB
- 9 FR
- 16 DE
- 22 NL
- 28 IT

English Français Deutsch Nederlands Italiano **ZONE 1**

Manufacturer:

Cochlear Bone Anchored Solutions AB Konstruktionsvägen 14, SE - 435 33 Mölnlycke, Sweden Tel: +46 31 792 44 00 Fax: +46 31 792 46 95

Regional offices:

Cochlear Ltd (ABN 96 002 618 073) 1 University Avenue, Macquarie University NSW 2109, Australia Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

Cochlear Americas 13059 E Peakview Avenue, Centennial, CO 80111, USA Tel: +1 303 790 9010 Fax: +1 303 792 9025

Cochlear AG EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland Tel: +41 61 205 0404 Fax: +41 61 205 0405

Local offices:

Cochlear Deutschland GmbH & Co. KG Karl-Wiechert-Allee 76A, D-30625 Hannover, Germany Tel: +49 511 542 770 Fax: +49 511 542 7770

Cochlear Europe Ltd 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey KT15 2HJ, United Kingdom Tel: +44 1932 87 1500 Fax: +44 1932 87 1526

Nihon Cochlear Co Ltd Ochanomizu+Motomachi Bldg, 2-3-7 Hongo, Bunkyo-Ku, Tokyo 113-0033, Japan Tel: +81 3 3817 0241 Fax: +81 3 3817 0245

Cochlear (HK) Limited Unit 1810, Hopewell Centre, 183 Queens Road East, Wan Chai, Hong Kong SAR Tel: +852 2530 5773 Fax: +852 2530 5183

Cochlear Medical Device (Beijing) Co Ltd Unit 2208-2212, Tower B, No.9 Gemdale Building, No.91 Jianguo Road, Chaoyang District, Beijing, PR China 100022

P.R. CHINA Tel: +86 10 5909 7800 Fax: (+86 10) 5909 7900

Cochlear Limited (Singapore Branch) 6 Sin Ming Road, #01-16 Sin Ming Plaza Tower 2, Singapore 575585 Tel: +65 6553 3814 Fax: +65 6451 4105

Cochlear Korea Ltd 1st floor, Cheongwon building, 828-5, Yuksam dong, Kangnam gu, Seoul, Korea Tel: +82 2 533 4663 Fax: +82 2 533 8408

Cochlear Benelux NV Schaliënhoevdreef 20 1, B - 2800 Mechelen, Belgium Tel: +32 1579 5511 Fax: +32 1579 5500

Cochlear Medical Device Company India (P) Ltd Platina Building, Ground Floor, Plot No. C-59, G-Block, BKC, Bandra (E), Mumbai – 400 0051, India

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Tel: +90 216 538 5900 Fax: +90 216 538 5919

Cochlear Canada Inc 2500-120 Adelaide Street West, Toronto, ON M5H 1T1 Canada Tel: +1 416 972 5082 Fax: +1 416 972 5083

www.cochlear.com

Welcome

Congratulations on your Cochlear™ Baha® Cordelle II Sound Processor. You are now ready to use Cochlear's advanced bone conduction sound processor.

This manual is full of tips and advice on how to best use and care for your Baha sound processor. By reading this manual and then keeping it handy for future reference, you'll ensure that you get the most benefit out of your Baha sound processor.

Key to device

See figure **1**

1. Body worn unit
2. Head worn unit
3. Direct audio input
4. Tone switch
5. Cord
6. Program switch (M/MT/T)
7. Microphone
8. Volume control, on/off
9. Electrical output
10. Plastic snap connector
11. Serial number
12. Electrical input
13. Clip
14. Battery compartment

Note on figures: The figures included on the front cover correspond to information specific to this model of sound processor. Please reference the appropriate figure when reading. The images shown are not to scale.

Introduction

The Baha® Cordelle II Sound Processor uses direct bone conduction to transmit sounds to the cochlea (inner ear). It works by combining a sound processor and a small titanium implant that is placed in the skull behind the ear. The skull bone integrates with the titanium implant through a process called osseointegration. This allows sound to be conducted via the skull bone directly to the cochlea, which improves hearing performance.

To ensure optimal performance, your hearing care professional will fit the processor to suit your needs based on your connection configuration. Be sure to discuss any questions or concerns that you may have regarding your hearing or use of this system with your hearing care professional. Be sure to follow all applicable instructions in this manual based on your specific connection.

Warranty

The warranty does not cover defects or damage arising from, associated with, or related to the use of this product with any non-Cochlear processing unit and/or any non-Cochlear implant. See "Cochlear Baha Global Limited Warranty card" for more details.

Contacting customer service

We strive to provide the best possible products and services to our clients. Your views and experiences of our products and services are important to us. If you have any comments you would like to share, please contact us.

Customer Service - Cochlear Americas

13059 E. Peakview Avenue, Centennial,
CO 80111, USA
Toll free (North America) 1800 523 5798
Tel: +1 303 790 9010 Fax: +1 303 792 9025
Email: customer@cochlear.com

Customer Service - Cochlear Europe

6 Dashwood Lang Road, Bourne Business Park,
Addlestone, Surrey KT15 2HJ, UK
Tel: +44 1932 87 1500 Fax: +44 1932 87 1526
Email: info@cochlear.co.uk

Customer Service - Cochlear Asia Pacific

1 University Avenue, Macquarie University, NSW
2109, Australia
Toll free (Australia) 1800 620 929
Toll free (New Zealand) 0800 444 819
Tel: +61 2 9428 6555 Fax: +61 2 9428 6352 or
Toll free 1800 005 215
Email: customerservice@cochlear.com.au

Key to symbols

Throughout the manual you will see small symbols. Please refer to the list below for explanations:

	"Consult instructions for use" or "Operating instructions"
	"Caution" or "Caution, consult accompanying documents"
	Audible signal
	CE-mark
	Temperature limitation
	Interference risk
	Batch Code
	Serial Number
	Refer to instructions/booklet

Getting started

Connecting the unit *See figure 1*

A plastic snap connector is mounted to the head worn unit. It is designed to snap onto the abutment and hold the head worn unit securely in place. The body worn unit is equipped with a clip so you can attach the unit to your clothes. The audio cord is connected by attaching the angulated plug to the body worn unit and the straight plug to the head worn unit. The three-pin plug is reversible.

On/Off *See figure 2*

Off ► When the volume control is in the 0 position it is off. You will feel a "click" as it goes into off mode.

On ► For the sound processor to be on, the volume control must be set to a position higher than 0. You will feel a click as the sound processor moves from off mode to on mode.

Volume control ► If you turn the volume control upwards the volume increases, and if you turn it downwards it will decrease.

Tone switch *See figure 3*

The tone switch controls the frequency response. The switch can be set in three different positions.

N = Normal. This gives the widest frequency response.

H = High frequency emphasis. This setting reduces low frequency sounds, which often can improve speech understanding.

L = Low frequency emphasis. This position is often useful during the first period after fitting as it allows you to gradually get used to high frequency sounds you have not heard in a long time.

The tone switch is effective for all inputs (microphone, telecoil and accessories connected via the direct audio input).

Program switch *See figure 2*

The body worn unit is equipped with a program switch for the telecoil function. The telecoil improves sound quality and speech understanding when using telephones in homes where there are personal loop systems installed and in other buildings with loop facilities.

There are three programs available. "M" is for listening with the microphone only. In program "M/T" both the microphone and telecoil are active. In program "T" only the telecoil is active.

Trim controls

There are two trim controls on the back of the body worn unit. These are to be adjusted by audiologists only. When fitting the Cordelle II, the audiologist will set these trimmers to suit your needs.

Telephones and mobiles *See figure 4*

Hold the telephone or mobile "upside down" close to the microphone (using the M program) or close to the telecoil (if using the MT or T program) as seen to the left. The telecoil is placed vertically, close to the electrical input. Please note the magnetic field from many modern phones is not strong enough to be picked up by the telecoil. Some phones can be equipped with a neck-loop to reinforce the magnetic field. It is important that neither you, nor the phone, touches the sound processor as this may cause feedback ("whistling").

Maintenance of your processor

The sound processor can be worn during waking hours apart from when you take a shower, or swim. Keep in mind that the sound processor consists of two delicate electrical devices that should be used with due care and attention. When engaging in physical activities, use common sense. If the activity is of such a nature that the sound processor risks getting damaged (for example contact sports) – remove it!

When not using your processor

When not in use ensure that the sound processor is switched off and stored carefully where the head worn unit and body worn unit will not gather dust and dirt. If you will not be using the sound processor for any length of time, remove the battery from the body worn unit. See figure 5

Attention: For children who have been fitted with a Baha sound processor it is extremely important that the parent or caregiver of the child provides all necessary assistance and support in cleaning and maintenance, as detailed above.

Batteries

The Cordelle II is powered by a 9 V battery. Three 9V NiMH rechargeable batteries (VARTA p7/8) are included with your Cordelle II and these should last for many years if handled with appropriate care. However, non-rechargeable batteries can also be used.

Battery replacement See figure 5

Open the battery compartment by pressing the cover at the bottom of the body worn unit and lift it up. Turn the body worn unit over and the battery will fall out. If it gets stuck you can shake the device carefully. Place the new battery with the + mark facing the + mark in the battery compartment. Push the battery in and close the cover.

 **Caution:** The Baha sound processor contains small parts that may be a risk for small children. Batteries can be harmful if swallowed. Be sure to keep your batteries out of reach of small children or people of diminished mental capacity. In the event a battery is accidentally swallowed, seek medical attention at the nearest emergency centre immediately.

Battery life

To maximise battery life, switch off the sound processor when not in use. Battery life varies depending on volume setting, the sound environment and the program setting. Battery life will decrease significantly when using the Baha sound processor with a connected FM receiver if the FM receiver uses the sound processor as its power source. Using powerful batteries is particularly important when high current consumption is common. The batteries included with the sound processor reflect our latest battery recommendation. Battery function can be related to a number of problems including: no sound, intermittent sound and crackling/buzzing. Thus, when encountering problems one of the first things one can do is to try changing the battery.

Rechargeable batteries

A fully charged battery will normally last for 1-2 days but this can vary considerably depending on settings and the sound environment. Do not forget to switch off the body worn unit when you are not using it to save the battery. Due to self-discharge, battery capacity always decreases over time. A fully charged battery will have about 80% of its capacity left after one month (60% after 6 months). After long storage, several charge/discharge cycles may be necessary to restore full capacity. Batteries have to be charged for 2-4 hours before use. A suitable battery charger can be purchased from Cochlear or a local electronics store. We recommend "intelligent" fast chargers for NiMH batteries.

Non-rechargeable batteries

Non-rechargeable batteries usually have a higher capacity and last longer. Batteries from different manufacturers tend to vary in size. Use only 9V alkaline batteries (max capacity 150mAh) and if possible, test the battery to make sure it fits and will function properly prior to purchase.

Caution: Non-rechargeable batteries cannot be recharged!

Note: Always follow local regulations for battery disposal.

General advice

- A sound processor will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic conditions.
- Infrequent use of a sound processor does not permit a user to attain full benefit from it.
- The use of a sound processor is only part of a hearing rehabilitation and may need to be supplemented by auditory training and instructions in lip-reading.
- The sound processor is suitable for use in electromagnetic environments with mains power of typical commercial or hospital quality, and power frequency magnetic fields of typical commercial or hospital levels. It is intended for use in an electromagnetic environment in which radiated RF (radio frequency) disturbances are controlled.
- Interference may occur in the vicinity of equipment the symbol to the right.

Accessories

Several accessories are available to enhance your sound experience in different contexts. To ensure the smooth operation of your Baha, only use Cochlear original accessories or accessories approved by Cochlear.

Direct audio input and microphone settings

The body worn unit is equipped with a direct audio input designed for connecting external equipment. The M/T switch automatically switches off the telecoil, if external equipment is connected, as well as regulating which sound sources are accessed (see below). Depending on the signal level, the volume control may have to be adjusted to achieve a normal listening level. If sound is distorted at a normal listening level, the electrical signal into the Cordelle II may be too high. If possible reduce the volume at the signal source so that the volume control on the Cordelle II can be left in a normal position.

M/T switch position	Direct audio input NOT connected	Direct audio input connected
M	Microphone only	Microphone only
M/T	Microphone and telecoil	Microphone and direct audio input
T	Telecoil only	Direct audio input only

Safety instructions when connecting external equipment

Only connection cables and accessories that are supplied or approved by Cochlear should be used. Non-standard connection cables can cause damage to the sound processor and injury to you. If in doubt, consult your audiologist. Cochlear is not liable for any damages arising as a result from this or any other form of incorrect use of the product or its parts.

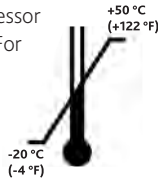
 **Caution: It is extremely dangerous and potentially fatal to connect any equipment that is plugged into a 110 and/or 220 V power supply without using the audio adapter.**

Warnings and precautions

- The sound processor is a digital, electrical, medical device designed for specific use. As such, due care and attention must be exercised by the user at all times.

- The sound processor is not waterproof! Never wear it in the bath or shower!

- Do not expose the sound processor to extreme temperatures etc. For example do not wear it in the sauna. The sound processor is designed to operate within the temperature ranges +5°C (+41°F) to +40°C (+104°F). In particular, battery performance deteriorates in temperatures under +5°C. The device should not be subject, at any time, to temperatures below -20°C (-4°F) or above +50°C (+122°F).

- This product is not suitable for use in the presence of flammable materials.
- If you are to undergo an MRI (Magnetic Resonance Imaging) procedure, refer to the MRI Reference Card included in the document pack.
- Dispose of electronic items in accordance with your local regulations.
- Do not dispose of in fire.

Troubleshooter

If you have encountered any problems with your Baha sound processor:

- Check that plastic housing is not touching anything (such as a hat, eyeglasses or similar items).
- Check the volume control.
- Check the program setting.
- Try changing the battery.

If the problem persists, contact your audiologist to get it fixed. Do not try fixing the sound processor yourself.

Service & repairs

If there is any fault with your Baha sound processor, contact your ear clinic or hearing aid centre and they will help you to get it serviced.

Telephone number of hearing centre: _____

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